



- Biocompatible-made of all natural Chitosan-based polymers
- Minimizes bleeding and edema after surgery
- Easily conforms to nasal anatomy
- Offers easier placement with a superior field of view
- No special storage conditions required

PRODUCT DESCRIPTION:

PosiSep® X is a hemostatic intranasal splint designed to minimize bleeding and edema and to prevent adhesions between the septum and the nasal cavity. **PosiSep® X** is placed in the nasal cavity after surgery or trauma. The **PosiSep®** line is constructed of a patient-comfortable sponge which is manufactured from naturally occurring Chitosan polymers. Subsequent irrigation will cause the product to fragment, thus optimizing patient tolerance and comfort.

- Fragments by way of gentle irrigation and aspiration
- Easy to apply-no mixing or additional prep time necessary

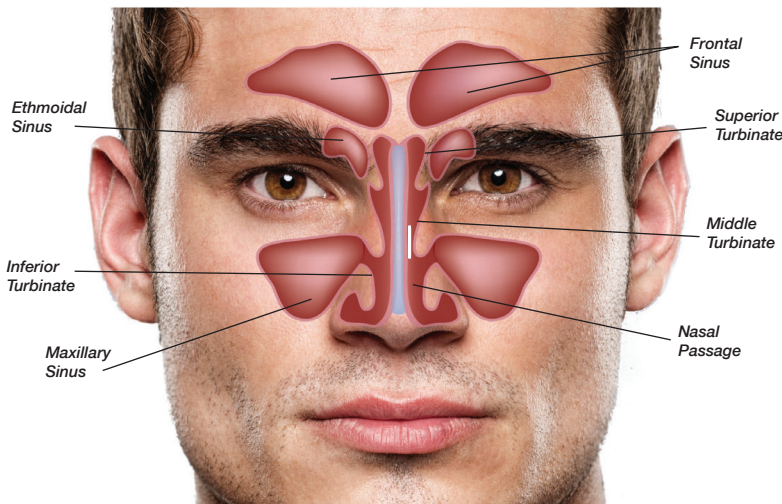


diagram not to scale

PosiSep® X Hemostat Dressing/Intranasal Splint is indicated for use in patients undergoing nasal/sinus surgery as a space occupying splint and hemostat to:

- Separate tissue or structures compromised by surgical trauma;
- Separate and prevent adhesions between mucosal surfaces during mesothelial cell regeneration in the nasal cavity;
- Help control minimal bleeding following surgery or trauma;
- Help control minimal bleeding following surgery or nasal trauma by tamponade effect, blood absorption and platelet aggregation; and
- Act as an adjunct to aid in the natural healing process.




PosiSep is indicated for use as a nasal hemostat to treat epistaxis.
PosiSep is intended for use under the direction of a licensed healthcare provider.

Before Hydration



After Hydration

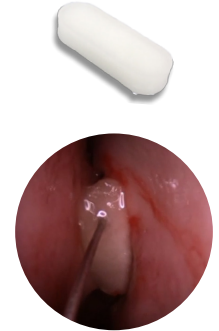

When PosiSep X is hydrated, it expands to fill the space preventing post-op adhesions.

What is Chitosan?

Chitosan (ky-toe-san) comes from a substance known as chitin, an amino-polysaccharide removed from the powdered shells of crustaceans.



For more information on PosiSep® products please contact your distributor

ORDERING INFORMATION

PosiSep® X is available in two sizes. See ordering information below.

Part #:	Description:
9215674	PosiSep® X Hemostat Dressing/Intranasal Splint 1.5 cm x 5.1 cm/ 8pk
9215664	PosiSep® X Hemostat Dressing/Intranasal Splint 1.3 cm x 3.8 cm/ 8pk

How Supplied:

PosiSep® X is packaged eight (8) sterile, single use units per carton. One (1) sterile unit contains one (1) Hemostat Dressing/Intranasal Splint. PosiSep® is terminally sterilized by gamma irradiation.